



Document Description: TSB- WAST – ProBP 3400 plugin 1.3.0.0 Release	Document Number: 20014145 Version: A
	Page 1 of 3
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.	

<h2 style="margin: 0;">Technical Service Bulletin</h2>			
Product:	WELCH ALLYN SERVICE TOOL v1.8.0.0, ProBP 3400 Devices	Date:	2016-01-28
Subject:	WAST ProBP 3400 plugin v.1.3.0.0		
HW Version(s) Affected:	N/A	SW Version(s) Affected:	1.8.0.0
Serial Numbers Affected:	N/A	Lot or Date Code Affected:	N/A
Classification:	Informational Only		
Distribution:	<input checked="" type="checkbox"/> Customer Care <input checked="" type="checkbox"/> Product Service <input checked="" type="checkbox"/> Field Service <input checked="" type="checkbox"/> ASPs <input checked="" type="checkbox"/> Distributors <input type="checkbox"/> Customers <input type="checkbox"/> Company Confidential		
Training Required:	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No		

Document Description: TSB- WAST – ProBP 3400 plugin 1.3.0.0 Release	Document Number: 20014145 Version: A
	Page 2 of 3
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.	

Summary:

The Welch Allyn Service Tool ProBP 3400 plugin v.1.3.0.0 is being released with enhanced functionality to support the ProBP 3400 device.

Note: With this release the service test box material number 407691 is no longer required to be used to perform a full calibration. Material will be Obsolete and made no longer for sale. All customers using a WAST version 1.7 or less should follow the below steps to update WAST to the latest version.

Also material 104125 SVC KIT, PROBP 3400 MOD F PCBA can now be Initialized from within the Calibration function of WAST and the MOD F PCBA can now be made available for sale to end users and service centers.

For detailed changes and fixes applied to this version of software please see the SRN document listed in the reference section.

Reference Documents

SWF: WAST 1.8.0.0	50011612
WAST Installer v.1.8.0.0 SRN	60069496
WAST Unresolved Anomalies Report	60051519
SWF: WAST ProBP 3400 Plugin 1.3.0.0	50011866
SRN: WAST ProBP 3400 Plug-in 1.3.0.0	60074502

Users should discontinue use of Versions 1.7.X.X and earlier of the Welch Allyn Service Tool (WAST).

Users running WAST Version 1.7.X.X and earlier must uninstall their current installed version of the Welch Allyn Service Tool, PartnerConnect (RSDS), and Welch Allyn USB driver from your PC using Add/Remove Programs before downloading and installing current Version 1.8.0.0.


Users currently running (or once they have updated to) WAST Version 1.8.0.0 will receive this plug-in update automatically when they next run WAST on an internet enabled PC.

Download the current released version of the Welch Allyn Service Tool and the Installation guide from the Welch Allyn web site <http://www.welchallyn.com/en/service-support/service-center/service-tool.html>

The ProBP 3400 plugin update will be loaded up in PartnerConnect and set to auto deploy to WAST 1.8.0.0 users only. Users that don't have WAST 1.8.0.0 will not receive this update until they upgrade WAST to the latest version.

Verification that the plugin has been deployed to the user's WAST installation can be made by opening WAST and going to the About Help screen and verify ProBP 3400 Plugin is at version 1.3.0.0

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.	
MPD FCD-1482 Rev 7	Page 2 of 3

Document Description: TSB- WAST – ProBP 3400 plugin 1.3.0.0 Release	Document Number: 20014145 Version: A
	Page 3 of 3
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.	

Software Compatibility

The following table identifies the Hardware/Software systems used in testing the ProBP 3400 Plugin v 1.3.0.0 for this release.

ProBP 3400 Device Firmware	1.03.05
Tested Operating Systems:	Windows 7x64 & 86 Windows 8x64 & 86 Windows 8.1x64 & 86
Welch Allyn Driver Set	a) WACP .NET Wrapper API (WelchAllyn.IO.dll) Version 2.11.4.25980 b) WACP CDLL (WACP.dll) Version 2.10.3.0 c) FileTransfer.dll Version 1.0.6.25981 d) RIFF.dll Version 1.0.0.25980 e) Upgrade.dll Version 2.3.1.25981 f) WAUSB driver 1.9.11

Version	Sec, Pg, Para Changed	Change Made	Date Version Created	Version Created By (initials)
A	N/A	Initial Release	2016-01-28	KMG

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.