



<b>Document Description:</b> VS100 SW V3.0.05.00 Release	<b>Document Number:</b> 80022377 <b>Version:</b> A
	
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.	

## Customer Service Bulletin

<b>Product:</b>	SPOT VISION SCREENER	<b>Date:</b>	2017-09-12
<b>Subject:</b>	SW V3.0.05.00 Release		
<b>HW Version(s) Affected:</b>	30028, 411370	<b>SW Version(s) Affected:</b>	All 3.0.xx.xx (below V3.0.04.06)
<b>Serial Numbers Affected:</b>	15001 and above for upgrading software  27419 and above for product manufactured in the US (411370)	<b>Lot or Date Code Affected:</b>	N/A

<b>Classification:</b>	As Needed		
<b>Distribution:</b>	<input checked="" type="checkbox"/> Customer Care	<input type="checkbox"/> Product Service	<input type="checkbox"/> Field Service
	<input type="checkbox"/> ASPs	<input checked="" type="checkbox"/> Distributors	<input checked="" type="checkbox"/> Customers
			<input type="checkbox"/> Company Confidential

<b>Training Required:</b>	<input type="checkbox"/> Yes	<input checked="" type="checkbox"/> No
<b>Summary:</b> Spot Vision Screener 3.0.05.00 software (SWF# 50012276) has been released for production and for product service in the US. Customers can also download this new software version on the Welch Allyn website.		
<b>Notable features include:</b>		
<ol style="list-style-type: none"> <li>1) <b>Software Enhancements</b>, continuous improvement to overall functionality.</li> <li>2) <b>Remote Web Service API for Exam Management</b>, this feature offers an easy, more automated way for 3rd party software programs to “order” data from Spot and “deliver” data back to their software program. This is a ReST API. ReST stands for Representational State Transfer and is a common architectural style that enables services to work best on the web.</li> </ol> <p>This web service interface can be used to develop additional custom tools for importing, exporting, and managing data on the Spot.</p>		

<b>Document Description:</b> VS100 SW V3.0.05.00 Release	<b>Document Number:</b> 80022377 <b>Version:</b> A
	
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.	

This open API is designed for smaller EMR and 3rd party software companies. This will NOT work for larger EMR companies (i.e. Cerner, Epic, AllScripts, etc.) To work with larger partners requires customization to their platform, which will be a future phase of connectivity.

Spot Vision Screener UI will look and feel the same. We will license the API and provide developer guides to partners identified by each regional team based on business opportunity. If you know of a potential/interested partner, please reach out to the Welch Allyn regional contact for Spot Vision Screener.

- 3) **Supports printing in all languages on the Brother QL720NW Printer**, Japanese, Chinese and all supported languages are now accurately represented and aligned on the label printer.
- 4) **Depleted coin cell battery requires date reset**, the updated software will prompt users to enter a date if the main board coin cell battery depletes, (not the main battery that powers the device).
- 5) **Addition of CFDA number on top screen for China**, this feature will identify product made in China for China. Only product manufactured in Suzhou, set in the Chinese language will see this revision.

This software update is for Spot Vision Screeners with a software version of **3.x or greater**

This CSB is to communicate Spot Vision Screener software V3.0.05.00 is now available and is available for customers to download and upgrade their devices on the Welch Allyn website.


Note: This upgrade is optional for customers however we recommend customers upgrade their device with the latest software version to optimize their experience.

**Software update procedure:** Please follow the software update instructions at the website (URL) below to download the file and upgrade the Vision Screener.

<https://www.welchallyn.com/content/welchallyn/americas/en/service-support/software-downloads.html>

Version	Sec, Pg, Para Changed	Change Made	Date Version Created	Version Created By (initials)
---------	-----------------------	-------------	----------------------	-------------------------------

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.

<b>Document Description:</b> VS100 SW V3.0.05.00 Release	<b>Document Number:</b> 80022377 <b>Version:</b> A
	
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.	

A	N/A	Initial Release	2017-09-12	MMS
---	-----	-----------------	------------	-----